

Keyboard and mouse shortcuts

The following list shows the default keyboard and mouse shortcuts in eUnity™.

For more information on customizing shortcuts, see the eUnity User Guide.

Modes

W	Toggle Window-Level Mode
Z	Toggle Pan / Zoom-Level Mode
N	Toggle Navigate Mode
T	Toggle Triangulation / Cursor Mode
L	Toggle Linear Measurement Mode
E	Toggle Elliptical ROI Measurement Mode
F	Toggle Free-form ROI Measurement Mode
A	Toggle Angle Measurement Mode
B	Toggle Cobb Angle Measurement Mode
C	Toggle Circle Measurement Mode
Y	Toggle Arrow Annotation Mode
G	Toggle Freehand Annotation Mode
Q	Toggle Series Repositioning Mode
V	Toggle Spine Labeling Mode

Flips and rotations

C + Shift	Rotate Series Clockwise
W + Shift	Rotate Series Counter-clockwise
H + Shift	Series Horizontally
V + Shift	Flip Series Vertically

Image/Series navigation tools

Cursor Key Up	Scrolls images from a higher number image to a lower number image
Cursor Key Down	Scrolls images from a lower number image to a higher number image

Keyboard and mouse shortcuts

Home	Returns the active View Box to the first image (lowest number image)
End	Returns the active View Box to the last image (highest number image)
Cursor Key Left	Place the previous series, which is not currently displayed, into the active viewport
Cursor Key Right	Place the next series, which is not currently displayed, into the active viewport

Mouse actions

Mouse Right Drag	Window Level Adjust
Mouse Middle Drag	Zoom with Mouse
Mouse Left Drag	<<current toolbar mode>>
Mouse Left Drag + Ctrl	Temporarily unlink series and scroll images
Mouse Right Drag + Ctrl	Pan with Mouse
Mouse Middle Drag + Ctrl	Triangulate with Mouse
Mouse Left Drag + Ctrl + Shift	Series reposition
Mouse Right Click	Show Context Menu

Study Tools

O	Revert study to original Window-Level and remove all markup
P	Toggle Cine Mode
D	Toggle Demographics
S	Toggle Reference Lines
K	Toggle Link Series
R	Show Report (if available)
M	Toggle Show / Hide Markup
SPACE	Toggle Cine Play / Pause

Window Level Presets

0 (zero)	Revert study to original Window Level
-----------------	---------------------------------------

Keyboard and mouse shortcuts

1 ... 9	Apply Window Level preset
----------------	---------------------------

Image Sharpening presets

0 (zero)	Revert sharpening level to original
1 ... 9 + Shift	Apply Window Level preset
S + Shift	Toggle Sharpening preset

Zoom tools

+	Zoom in on the active viewport
-	Zoom out on the active viewport

Markup

Delete	Deletes the selected markup
---------------	-----------------------------

MPR / 3D

O + Shift	Create oblique
B + Shift	Create curved MPR
I + Shift	Toggle MPR curve orientation indicator
L + Shift	Toggle linked MPR curve orientation indicators
R + Shift	Reset MPR curve rotation angle
C + Shift	Toggle clip box
P + Shift	Cut plane mode
F + Shift	Scalpel mode
Esc	Cancel segmented area
W + Shift	Settings panel